PRODUCT NAME: DR WOUND DEBRICLEAR GEL
PRODUCT SIZE(ml): 15ml,20ml,30ml,50ml,60ml.

Dr Wound Debriclear Gel is a gel with the main component of sodium alginate and sodium hyaluronate. This product is suitable for removing necrotic and slough tissues autolytically without damaging the viable healing tissue.

Active Ingredient:

Sodium alginate and sodium hyaluronate.

Intended use:

Dr Wound Debriclear Gel is intended for removing necrotic and slough tissues autolytically without damaging the viable healing tissue. It can be used for all types of wounds with necrotic or slough tissue.

Indication:

All types of wound with necrotic or slough tissue.

Features and Benefits:

- Moisturizing dry, necrotic tissues
- Non invasive
- Selective autolytic debridement with no pain and bleeding
- Biocompatible

How To Use:

- 1.Clean wound area with Dr Wound Nano Silver Spray or other appropriate cleansing solutions. Do not dab dry.
- 2.Place Dr Wound Debriclear Gel on desired area.
- 3.Cover it with appropriate non-adherent absorbent dressing.
- 4.Fix it with appropriate surgical dressing.

Storage:

Store in a cool, dry place at temperature below 30°C. Keep away from direct sunlight.

Warning:

- Dr Wound Debriclear Gel is intended for external use on skin and wounds. Avoid contact with eyes, mucous membranes, and internal body cavities. If contact occurs, rinse immediately with water.
- Dr Wound Debriclear Gel should never be ingested. If swallowed, seek medical attention immediately.
- Do not use if the seal is broken or the product appears damaged.
- If irritation, redness, or any adverse reaction occurs, discontinue use and consult a healthcare professional.

Contraindications :

Dr Wound Debriclear Gel should not be used in patients with known allergies to sodium alginate, sodium hyaluronate, or other components.

The gel is also not intended for use in the eyes or mucous membranes unless directed by a healthcare professional.

Manufactured and Distributed by :
Dikilang dan Diedarkan oleh :

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