PRODUCT NAME : DR WOUND NANO SILVER SPRAY
 PRODUCT SIZE (ml) : 20ml,30ml,50ml,100ml & 150ml

Dr wound Nano Silver Spray contains Water and Nano Colloidal Silver.

Based on breakthrough in nanotechnology, it is indicated for rinsing and cleansing spray solution for cleaning all types of wounds.

Active ingredient :

Nano Colloidal Silver

Functions:

Effectively kills more than 650 microorganism (bacteria, fungus, and virus)

Non Alcohol

Non-irritant

Storage Condition:

Store at room temperature.

Keep the packaging tightly closed and away from heat sources.

Indication:

It is suitable for all types of wound. Minor cuts and scratches, first and second degree burns, abrasion, post-surgery wound, episiotomy, cesarean wound, circumcision, diabetic ulcer and first aid.

Contraindications:

Dr Wound Nano Silver Spray should not be used on person with a known allergy or sensitivity to any of its components or ingredients.

Direction:

1. Wash hands thoroughly before and after use.
2. Prime the spray with 4 or 5 pumps.
3. Hold the spray approximately 5-10cm from the area requiring cleansing. An area of about 10x10cm will require about 5 sprays to cover. If the wound contains a lot of debris, slough, is at risk of infection and/or requires irrigation more sprays will be necessary.
4. Allow Dr Wound Nano Silver Spray to dry naturally, this may take up to about a minute.
5. Cover with a secondary dressing suitable for the wound type.
6. Repeat application at every dressing change.
7. Do not dilute.

Cautions:

- Dr Wound Nano Silver Spray is for single patient use only.
- Do not use if the product packaging is open or damaged.
- Use on paediatrics and breast feeding women under medical supervision.
- Product should be disposed in waste bins properly.
- Do not use if skin reaction observed.
- Avoid contact with eyes.

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PRODUCT NAME : DR WOUND BIOHEAL GEL

PRODUCT SIZE (ml) : 7ml, 20ml & 60ml

Dr Wound BioHeal Gel provides moist environment and attracts necessary cells including proteins for faster wound healing.

Active ingredient :
Chitosan Biopolymer

Functions:
Promotes healing in all types of wounds & burns.

Storage Condition:
Store at (5-30°C).
Keep away from moisture and direct sunlight.

Indication:
It is suitable for dry to moderate exuding wound. Minor cuts and scratches, first and second degree burns, abrasion, post-surgery wound, episiotomy, cesarean wound, circumcision, diabetic ulcer and first aid.

Contraindications:
Dr Wound BioHeal Gel should not be used on person with a known allergy or sensitivity to any of its components or ingredients.

Direction:
Apply thin layer of Dr Wound BioHeal Gel on the affected area.
Cover area with secondary dressing if necessary.

Cautions:

- For external use only.
- Keep out of reach of children.
- Do not use if skin reaction observed.
- Product should be disposed in waste bins properly.
- Do not use if the product packaging is open or damaged.

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Manufactured and Distributed by :
Dikilang dan Diedarkan oleh :

FARMASIA

Advanced Care Today

Farmasia Sdn Bhd (772557-D)
B2-2 Ribosom Block,
UKM-MTDC Technology Centre,
43650, Bandar Baru Bangi,
Selangor, Malaysia.