PRODUCT NAME : DR WOUND NANO SILVER SPRAY

PRODUCT SIZE (ml) : 50ml & 100ml

**Dr wound Nano Silver Spray** contains Water and Nano Colloidal Silver.

Based on breakthrough in nanotechnology, it is indicated for rinsing and cleansing spray solution for cleaning all types of wounds.

**Active ingredient :**

Nano Colloidal Silver

**Functions:**

Effectively kills more than 650 microorganism (bacteria, fungus, and virus)

Non Alcohol

Non-irritant

**Storage Condition:**

Store at room temperature.

Keep the packaging tightly closed and away from heat sources.

**Indication:**

It is suitable for all types of wound. Minor cuts and scratches, first and second degree burns, abrasion, post-surgery wound, episiotomy, cesarean wound, circumcision, diabetic ulcer and first aid.

**Contraindications:**

Dr Wound Nano Silver Spray should not be used on person with a known allergy or sensitivity to any of its components or ingredients.

**Direction:**

1. Wash hands thoroughly before and after use.

2. Prime the spray with 4 or 5 pumps.

3. Hold the spray approximately 5-10cm from the area requiring cleansing. An area of about 10x10cm will require about 5 sprays to cover. If the wound contains a lot of debris, slough, is at risk of infection and/or requires irrigation more sprays will be necessary.

4. Allow Dr Wound Nano Silver Spray to dry naturally, this may take up to about a minute.

5. Cover with a secondary dressing suitable for the wound type.

6. Repeat application at every dressing change.

7. Do not dilute.

**Cautions:**

- Dr Wound Nano Silver Spray is for single patient use only.

- Do not use if the product packaging is open or damaged.

- Use on paediatrics and breast feeding women under medical supervision.

- Product should be disposed in waste bins properly.

- Do not use if skin reaction observed.

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Manufactured and Distributed by:  
Diklikang dan Didistributikan oleh:

Advanced Care Today

Farmasia Sdn Bhd (772557-D)  
B2-2 Ribosom Block,  
UM-MATC Technology Centre,  
43650, Bandar Baru Bangi,  
Selangor, Malaysia.

PRODUCT NAME : DR WOUND BIOHEAL GEL

PRODUCT SIZE (ml) : 7ml, 20ml & 60ml

**Dr Wound BioHeal Gel** provides moist environment and attracts necessary cells including proteins for faster wound healing.

**Active ingredient :**  
Chitosan Biopolymer

**Functions:**  
Promotes healing in all types of wounds & burns.

**Storage Condition:**  
Store at (5-30°C).  
Keep away from moisture and direct sunlight.

**Indication:**  
It is suitable for dry to moderate exuding wound. Minor cuts and scratches, first and second degree burns, abrasion, post-surgery wound, episiotomy, cesarean wound, circumcision, diabetic ulcer and first aid.

**Contraindications:**  
Dr Wound BioHeal Gel should not be used on person with a known allergy or sensitivity to any of its components or ingredients.

**Direction:**  
Apply thin layer of Dr Wound BioHeal Gel on the affected area.  
Cover area with secondary dressing if necessary.

**Cautions:**

- For external use only.
- Keep out of reach of children.
- Do not use if skin reaction observed.
- Product should be disposed in waste bins properly.
- Do not use if the product packaging is open or damaged.

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Manufactured and Distributed by :  
Dikilang dan Diedarkan oleh :

**FARMASIA**

Advanced Care Today

Farmasia Sdn Bhd (772557-D)  
B2-2 Ribosom Block,  
UKM-MTDC Technology Centre,  
43650, Bandar Baru Bangi,  
Selangor, Malaysia.