

# ATROFLEX™

GEL

## THE GEL FOR JOINTS AND MUSCLES

PRODUCT NAME : ATROFLEX GEL

PRODUCT SIZE (ml) : 50ml & 100ml

- Extra virgin ozonised olive oil.
- Eco-friendly active ingredient.
- Non-invasive, localised application.
- Suitable for continuous & long term application.
- Safe to use.
- Can complement oral supplement or painkillers.

ATROFLEX™ GEL contains higher oxygen concentration than most products to ensure optimal efficacy and co-formulated with jojoba oil to improve absorption rate. It is registered as a medical device and used in many rehabilitation centres. ATROFLEX™ GEL's ozonised olive oil is produced with a patented process which makes it lighter, more stable and improved skin penetration. It is also gives slight cooling sensation on the skin while the active ingredient penetrate the targeted area.

### INGREDIENTS :

Extra Virgin Ozonised Olive Oil\*, Purified Water, Propylene Glycol, Carbomer, Sodium Citrate, Polyacrylate Crosspolymer-6, Mentha Piperita (Peppermint) Oil, Phenoxyethanol, Sodium Chloride, Disodium (EDTA), Sodium Benzoate, D-Limonene, Linalool, Simmondsia Chinensis (Jojoba) Seed Oil.

\*Ozone carries a unique and pungent smell. It is a sign of active ingredient's authenticity.

### INTENDED USE :

ATROFLEX™ Gel helps to reduce pain in knees, back, shoulder, other joints, and muscles.

### STORAGE CONDITION :

Store in a dry place at temperature between 5°C to 25°C. Keep away from moisture and direct sunlight or heat sources.

### PRECAUTIONS :

Keep out of reach of children.  
ATROFLEX™ Gel is intended for external use only.  
Read instruction before use.

### WARNING :

Do not use if there is known allergic reaction to any of the ingredients.  
Do not use if security seal is broken.  
No study was done to show any contraindications on pregnant or breastfeeding women.

There is no special requirement for waste disposal.  
Dispose properly in waste bin.

# ATROFLEX™

GEL

## THE GEL FOR JOINTS AND MUSCLES

### HOW TO USE :



1. Squeeze 1 to 2 inches of ATROFLEX™ GEL on your finger.



2. Apply the gel on the affected area and massage gently until the gel is fully absorbed.

3. Apply 2 or 3 times a day depending on the degree of pain.

4. Apply before doing any physical activities to reduce the risk of muscle injuries and joint pain.

Characteristics	Ozonised Olive Oil in ATROFLEX™ GEL	Explanation
Production process temperature	9°C to 12°C	Low temperature avoids degradation and denaturation of oil. Degradation and denaturation of oils often leads to production of unwanted by products that may be harmful to our body.
Manufacturing Process	Patented manufacturing process (Peroxibikey®)	Peroxibikey® production of ozonised olive oil with better and consistent quality.
Ozone saturation or peroxide value	High PV (peroxide value) of 600 MeqO <sub>2</sub> / kg	Higher PV means more oxygen molecules are saturated into the olive oil. Most ozonised olive oil only holds up to 250 MeqO <sub>2</sub> / kg of PV value due to the limitation in manufacturing process (more harmful by-products may be traced to achieve higher PV values).
Raw material	100% natural from Spain (the biggest producer of olive oil in the world)	Premium quality olive oil
Green technology	Uses certified organic raw material with Eco-Cert	Product is Eco-friendly and sustainable.
Halal	Halal certified	Halal means that the product is made from allowed ingredients and best hygiene practice according to Muslim's belief.

Manufactured and Distributed by :



Advanced Care Today  
Farmasia Sdn Bhd (772557-D)  
62-2 Ribson Block  
UKM-MTDC Technology Centre,  
43650, Bandar Baru Bangi,  
Selangor, Malaysia.

### MEDICAL DEVICE

Registered under Act 37

MDA No. : GA4424120-47522

Product Code: FFP060.030(30ml)/FFP060.050(50ml)/FFP060.100(100ml)